

Action[®] Products Recommended Disinfectants Chart

Below are the cleaning products recommended for use with Action[®] pads and positioners using the spray-wipe-rinse cleaning protocol. Products must be thoroughly rinsed with clear water after cleaning to remove any cleanser residue. Pads are washable at temperatures of approximately 103°F/ 40°C. Gas sterilization, autoclaving, machine washing and drying are not recommended for these products as it can damage the product. Soaking is not recommended as it may seriously compromise product integrity, making it inappropriate for continued use. Please call or email Customer Service with any questions or to receive our complete Care and Use Guidelines, 800.228.7763/ 301.797.1414 or service@actionproducts.com.

Reagent Name	Active Ingredient	Recommended
2030 Bio-Cidol Neutral	Dymethyl Ammonium Chloride	Yes
Accel [®]	Accelerated Hydrogen Peroxide	Yes
Antifect [®] AF	Dymethyl Ammonium Chloride	Yes
Betadine [®]	Povidone-Iodine	Yes
Bruphene 256	Amylphenol	Yes
Buraton [®] 10F	Dialdehyde	Yes
Clorox [®] Bleach	Sodium Hypochlorite, 6.0%	Yes
Defend [®] PS-10	Dymethyl Ammonium Chloride	Yes
Dispatch [®]	Sodium Hypochlorite, 1.0%	Yes
Endbac [®]	Dymethyl Ammonium Chloride	Yes
Hibiclens [®] Liquid	Chlorhexidine Gluconate	Yes
Sani-Cloth [®] Plus	Dymethyl Ammonium Chloride	Yes
Phenex NS [™]	Amylphenol	Yes
Quat-64 [™] Broad	Dymethyl Ammonium Chloride	Yes
TB-Cide Quat [®]	Quaternary Ammonium	Yes
Terralin [®]	Dymethyl Ammonium Chloride	Yes
Vesphene [™] LpH se	Phenylphenol	Yes
Virex Tb [™]	Dymethyl Ammonium Chloride	Yes
CaviCide [®]	2-Propanol (isopropyl alcohol)	Not Recommended
Endozime [®] AW	2-Propanol (isopropyl alcohol)	Not Recommended
Mikrozid [®] Liquid	2-Propanol (isopropyl alcohol)	Not Recommended

Action[®] and Making a Material Difference[®] are registered trademarks of Action Products, Inc. USA.