

# ACTION® Action® O.R. Products - Resurfacing (Repair)

 **Before you start your repair, please read the instructions below.**


- Consult your individual facility's protocol for resurface or repair of any product.
- Products with foam and fabric are not recommended for resurfacing. If damaged, we recommend they be removed from service, discarded and replaced immediately.
- Polymer products may be resurfaced if they have minor cuts or light abrasions only. Multiple attempts to resurface a single product are not recommended. Presence of tears, gouges, large cracks, bubbles, or excessively deep creases indicates a product should be removed from service, discarded and replaced.
- Inspect any resurfaced product before each use to ensure the film resurfacing tape remains secure and clean. There should be no loose edges or presence of debris.

## Action® Resurfacing Kits

Two sizes of resurfacing kits are available for use.

KIT002 – Resurfacing Patch: 4" x 4"

KIT003 – Resurfacing Roll: 36" x 4"

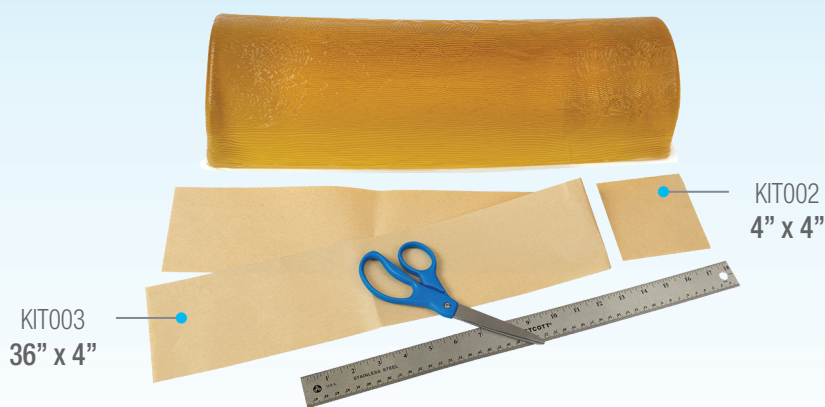

## Warranty Info

### Action® O.R. Support Surfaces & Patient Positioners

- Support surfaces and patient positioners are warranted to be free of defective materials and workmanship at the time of purchase and are serviceable for a period of two years provided the product is not subject to misuse, negligence, abuse, alteration, or failure to follow the Care and Use instructions.
- Accessories are warranted to be free of defective materials and workmanship at the time of initial purchase only.

**Action Products, Inc.**

800.228.7763 • 301.797.1414 • Fax 877.733.2073

[www.actionproducts.com](http://www.actionproducts.com) • [service@actionproducts.com](mailto:service@actionproducts.com)

ACTION and AKTON are registered trademarks of Action Products, Inc. © 2025 Action Products, Inc.






### Step 1:

Clean and disinfect according to the [OR IFU: Care and Use Guide](#) prior to resurfacing the product.

[https://www.actionproducts.com/wp-content/uploads/2023/08/ACTION-OR-IFU\\_English.pdf](https://www.actionproducts.com/wp-content/uploads/2023/08/ACTION-OR-IFU_English.pdf)

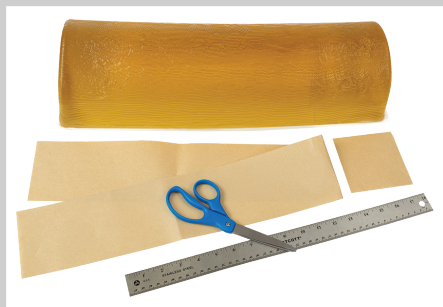

### Step 2:

If necessary, cut film tape to proper size, allowing a 0.25 – 0.5" (0.6 – 1.3cm) border of film to ensure damaged area is fully encased beneath the film.



### Step 3:

Carefully peel back one-half of release paper and crease it.



### Step 4:

Carefully adhere exposed portion of film tape to damaged surface, pushing out any air bubbles that may become trapped beneath the film tape.



### Step 5:


Continue to apply tape while slowly removing the release paper until film tape is firmly secured to the product surface.



### Step 6:

Clean and disinfect product before placing it into service.

See Action® products resurfacing video at:  
<https://www.actionproducts.com/resources/product-cleaning-and-resurfacing/or-product-resurfacing/>

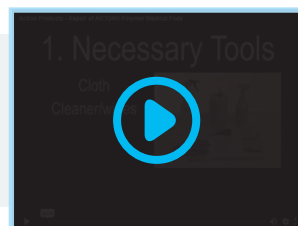

Action Products, Inc.

800.228.7763 • 301.797.1414 • Fax 877.733.2073

[www.actionproducts.com](http://www.actionproducts.com) • [service@actionproducts.com](mailto:service@actionproducts.com)

ACTION and AKTON are registered trademarks of Action Products, Inc. © 2025 Action Products, Inc.



MADE IN  
USA



M8009-000